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Eye drops with sodium hyaluronate 0.15%

DESCRIPTION

HYALOVET® cares for and moisturizes the ocular surface and alleviates the symptoms of dry eye syndrome (e.g. irritated, red, itchy, watery and/or very sensitive ocular surface) by forming a preservative-free, viscoelastic, clear protective film on the cornea without impairing visual performance. This is achieved by the high water-binding capacity of the sodium hyaluronate contained, the sodium salt of hyaluronic acid, which also occurs naturally in the vitreous body of the eye, among other things.

As a result, **HYALOVET®** causes a long-lasting relief, moisturizing and care from discomforts associated with:

- mechanical stress, e.g. during wearing of hard or soft contact lenses, or during ophthalmic diagnostic procedures
- environmental stress, e.g. induced by air conditioning, wind, coldness, dryness or air polluted by cigarette smoke for example
- work on the computer, on microscopes and during/after long car journeys
- regular use of certain medicines
- decreasing tear production with age
- allergens such as pollen, animal hair, dust etc.

Preservative free **HYALOVET®** is also well-tolerated during long-term use.

HYALOVET® is suitable for contact lenses wearers. The contact lenses do not have to be removed during the application of the eye drops.

APPLICATION

Detach one single dose container. Make sure that each single dose container is intact. To open the single dose container, twist off the flat end (do not pull). Tilt your head back a little, look upwards and gently pull the lower eyelid away from the eye and instill one drop in the conjunctival sac of the eye 3 - 5 times daily. Close your eyes slowly to allow an even distribution of the liquid on the eye surface. If the discomfort gets worse or does not improve, please consult your ophthalmologist!

COMPOSITION

1 ml contains: sodium hyaluronate 0.15%, sodium chloride, sodium citrate, citric acid, water for injections.

PACKAGE SIZES

Commercial units: single dose containers, 1 x 0.35 ml / 2 x 0.35 ml / 5 x 0.35 ml / 10 x 0.35 ml / 15 x 0.35 ml / 20 x 0.35 ml / 30 x 0.35 ml / 60 x 0.35 ml / 100 x 0.35 ml
Not all pack sizes may be marketed.

PRECAUTIONS AND WARNINGS

- Do not use the product if single dose container is damaged.
- Do not touch the eye with the single dose container.
- Do not use if you are allergic to any of the ingredients.
- In very rare cases, the use of the eye drops may cause irritation in very sensitive eyes.
- For single use only; as **HYALOVET®** does not contain preservatives, any solution not used immediately after opening, should be discarded.
- If you also use other eye drops / eye ointment, wait about 15 minutes. **HYALOVET®** should always be used last.
- Do not use in infected or injured eyes.
- Temporary blurring for a brief period may be experienced after application; avoid driving, operating machinery and wait until normal vision is restored.
- Do not use the eye drops beyond the expiry date which you can find on each single dose container, and on the folding box.
- **HYALOVET®** is in small plastic containers, which could be swallowed by small children. Therefore, please keep the product out of the reach of children.
- In case a serious incident has occurred during the use of this medical device or as a result of its use, please report it to the manufacturer and the competent authority.

STORAGE

Store at 2° C to 25° C. Do not freeze. Protect from light by storing the single dose containers in the folding box.

EXPLANATION OF THE ICONS

 Batch code

 Expiry date

 Date of manufacture

 Sterilized using aseptic processing techniques

 Single sterile barrier system with protective packaging outside

 Consult instructions for use

 Temperature limit and storage temperature

 Keep away from sunlight, store the single dose container from light in the original carton

 For one single use only

 Do not use if package is damaged (single dose container) and consult instructions for use

 Medical device

 Unique device identifier

 Manufacturer

 Distributor

 CE marking = symbol of free marketability in the EU with indication of the involved Notified Body

 **Manufacturer:**
Penta Arzneimittel GmbH
Werksstraße 3
92551 Stulln, Germany

 **Distributor for Greece and Cyprus:**
Zwitter Pharmaceuticals LTD
Pentelis Ave 34-36
152 34 Chalandri, Greece
T: +30 210 6821098
E: zwitter@zwitter.gr
W: www.zwitter.gr

 0483