Lidblef®

DAILY CARE

EYELID CLEANSING FOAM FOR OCULAR SECRETIONS

EN For external use only
COMPOSITION

Terpinen-4-ol, Anthemis Nobilis (Chamomile) Essential Oil, D-Panthenol, Allantoin and purified Water.

INDICATIONS

If you have dry, scratchy, irritated eyes the problem may not be your eyes, but rather your eyelids.

LIDBLEF® DAILY CARE is designed to treat irritated eyelids affected by pathologies such as dryness, allergies, blepharitis and blepharoconjunctivitis due to the presence of Demodex (eyelid mites) and/or bacterial infestations.

The effect of **LIDBLEF® DAILY CARE** is obtained through the mechanical removal of excessive oil, pollen, and other debris from the eyelids.

In case of persisting secretions and residues, daily application is recommended.

This foam can also be used to clean the base of the eyelashes and eyelid margins before and after surgery.

LIDBLEF® DAILY CARE has an improved formula that makes it suitable to be used as maintenance care, for patients who have followed the blepharitis treatment with **LIDBLEF® INTENSIVE CARE**.

DESCRIPTION

LIDBLEF® DAILY CARE contains **Terpinen-4-ol**, one of the components of tea tree oil, which, thanks to its oily texture, helps to eliminate secretions of eyelids and eyelashes.

LIDBLEF® DAILY CARE contains Allantoin to help keep the skin hydrated.

WHAT IS BLEPHARITIS?

It is a chronic inflammation of the eyelids and is one of the most common disorders of the eye, usually due to the presence of Demodex (eyelid mites) and bacterial infestations. This is often the main cause of eye discomfort, redness and tearing.

Other eye symptoms of blepharitis include burning, itching, light sensitivity, irritation, sandy and gritty sensation that are more accentuated upon awakening.

Blepharitis is a chronic disease for which there is no cure and requires a long-term treatment, hygiene and maintenance to keep it under control and prevent subsequent related complications linked to this type of conditions.

WHAT IS DRY EYE?

It is known as a disorder of the tear film due to tear deficiency or excessive tear evaporation, which can cause damage to the interpalpebral and ocular surface.

IN CASE OF EYE SURGERY

Before or after eye surgery, there is an increased risk of infections on the eyelids and eyelashes, so it is important to follow the recommendations of your eye specialist with regard to the hygiene and care to follow.

PACKAGING

50 ml

REF F50LD493

5 ml (sample)

REF F5LD495

PROPERTIES

- **LIDBLEF® DAILY CARE** is a non-greasy and non-irritant foam formulation, for topical use.
- **NOT TESTED ON ANIMALS.**

INSTRUCTIONS FOR USE

- 1 - Before the application of the foam, wash your hands and then clean the fingertips with **LIDBLEF® DAILY CARE**.
- 2 - Press the delivery pump and place the foam on the clean fingertips. When used before or after surgery it is recommended to place the foam on sterile gauze.
- 3 - Close the eye and massage the eyelid and eyelashes with **LIDBLEF® DAILY CARE**. Do not touch or open the eyes during the application. Leave the foam on for approximately 60 to 80 seconds.
- 4 - After each application rinse the eyelashes and eyelids with warm water.

Note: Repeat the same steps for the other eye.

Repeat this treatment 2 - 3 times a day for an efficient maintenance care of eyelids and eyelashes.

PRECAUTIONS

- Always wash your hands before and after use.
- Remove contact lenses before the application of the foam. Ensure all foam and residues are removed before re-inserting lenses.
- Keep your eyes closed during the application.
- The product is intended for the application on eyelids and eyelashes.
- **Do not apply directly in the eyes.**
- **For external use only.**
- Keep out of the sight and reach of children.
- Don't use the product in children under 6 years old. For children over 6 years old the application must be performed by an adult and only after your specialist's advice.
- In case of use in pregnant or lactating women it is advised to consult your specialist beforehand.
- Use hypoallergenic eye makeup and delicate skin lotions around your eyes during use.
- Do not use in case of known hypersensitivity to one of the constituents of the product.
- If a reaction to the product occurs, stop use immediately, rinse your eyes with water and contact your eye specialist if the symptom continues.
- Discard the remaining solution 3 months after first opening.
- Do not use if the bottle is damaged.
- **Do not use after the expiry date.**

STORAGE

- Store between 5°C and 35°C and protect from a direct source of light.

Zwitter
IMPORTER
ZWITTER PHARMACEUTICALS LTD
Aesop, Pervélengos 34-36,
152 34, Χαλάνδρι, Αθήνα, Ελλάδα
Tηλ.: +30 210 6821098
E-mail: zwitter@zwitter.gr
www.zwitter.gr

NOVAX® PHARMA
VITEVAX® PHARMA
Le Coronado
20, Av. de Fontvieille
MC 98000 MONACO
www.novaxpharma.com

CE
0051

V1 - Rev. 14/07/2020

PRESENTATIONS: LIDBLEF® is presented in 2 formulations:

- **LIDBLEF® INTENSIVE CARE** to eliminate severe ocular secretions and debris due to infections caused by bacteria and Demodex (red box).
- **LIDBLEF® DAILY CARE** for the maintenance care of eyelids and eyelashes (blue box).

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