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Eye drops with sodium chloride

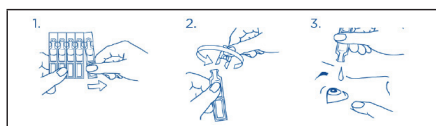
DESCRIPTION

SALZ 5% is used for treatment of corneal oedema and removes, due to its physical-osmotic properties, the excess liquid and thus improves vision.

SALZ 5% is applied especially after eye surgeries (e.g. cataract surgery) and after diagnostic interventions on the eye (e.g. gonioscopy), if there is a malfunction of the osmotic balance of the corneal tissue.

APPLICATION

Detach one single dose container. Make sure that each single dose container is intact (1). To open the single dose container, twist off the flat end (do not pull) (2). Tilt your head back a little, look upwards and gently pull the lower eyelid away from the eye and instill 1 - 2 drops in the conjunctival sac of the eye 3 - 5 times daily (3). Close your eyes slowly to allow an even distribution of the liquid on the eye surface.

If the discomfort gets worse or does not improve, please consult your ophthalmologist.

COMPOSITION

Sodium chloride, water for injections.

PACKAGING SIZES

Commercial units: 20 x 0.5 ml, 50 x 0.5 ml

Not all pack sizes may be marketed.

PRECAUTIONS AND WARNINGS

- Do not use without medical recommendation!
- Do not use the product if single dose container is damaged.
- Do not touch the eye with the single dose container.
- Do not use if you are allergic to any of the ingredients.
- In this type of corneal diseases the wearing of contact lenses should be avoided in general. Should it be mandatory to wear contact lenses, so remove them before applying. Wait after the application at least 15 minutes before inserting the contact lenses again.
- Directly after application, a burning sensation and/or irritation of the affected eye may occur.
- For single use only; as SALZ 5% does not contain preservatives, any solution not used immediately after opening should be discarded.
- If other eye preparations are to be applied to the eye at the same time as **SALZ 5%**, please consult your treating ophthalmologist. An interval of at least 15 minutes should be maintained between the two applications.
- Do not use in infected eyes.

- Temporary blurring for a brief period may be experienced after application; avoid driving, operating machinery and wait until normal vision is restored.
- Do not use the eye drops beyond the expiry date which you can find on each single dose container and on the folding box.
- **SALZ 5%** is in small plastic containers which could be swallowed by small children. Please keep the product out of the reach of children.
- In case a serious incident has occurred during the use of this medical device or as a result of its use, please report it to the manufacturer and the competent authority.

STORAGE

Store at 2° C to 25° C. Do not freeze. Protect from light by storing the single dose containers in the original folding box.

	Batch code
	Expiry date
	Date of manufacture
	Sterilized using aseptic processing techniques
	Single sterile barrier system with protective packaging outside
	Consult instructions for use
	Temperature limit and storage temperature
	Keep away from sunlight, store the single dose container protected from light in the original folding box.
	Do not use if package is damaged (single dose container) and consult instructions for use
	For one single use only
	Medical device
	Unique device identifier
	Manufacturer
	Distributor

CE 0483 CE marking = symbol of free marketability in the EU with indication of the involved Notified Body

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Penta Arzneimittel GmbH Werksstraße 3, 92551 Stulln, Germany
Zwitter Pharmaceuticals LTD