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### HyloPan (MONODOSE)

Eye drops with sodium Hyaluronate 0,15% and Dextranthenol 2%

### DESCRIPTION

HyloPan (MONODOSE) is a sterile, preservative-free, elastostomous, clear protective viscoelastic film for the cornea. HyloPan contains Sodium Hyaluronate, the sodium salt of hyaluronic acid, which forms due to its physical properties, an evenly, solid and long-retaining protective shield to the eye's epithelial layer without impairing vision. In addition, HyloPan contains provitamin B5 (Dextranthenol), which has the ability to bind a high quantity of water, moisturizes and care the eye, on the one hand, and on the other hand, supports the Sodium Hyaluronate to hold moisture.

The combination of Sodium Hyaluronate (tear film stabilizing) and Dextranthenol (nourishing, calming) protects and moisturizes the eye surface and lubricates it.

As a result HyloPan causes long-lasting relief from discomforts associated with:

- mechanical stress, e.g. during wearing of hard or contact lenses, or during ophthalmic diagnostic procedures,
  - environmental stress, e.g. induced by air conditioning, wind, coldness, dryness or air polluted by cigarette smoke for example,
  - visual stress from intensive viewing, e.g. at computer terminals, microscopes or during prolonged operation of motor vehicles.
- Preservative free HyloPan is also well-tolerated during long-term use.
- HyloPan is suitable for contact lenses wearers. The contact lenses do not have to be removed during the application of the eye drops.

### APPLICATION

Detach one single dose container. Make sure that each single dose container is intact. To open the single dose container twist off the flat end (do not pull). Tilt your head back a little, look upwards and gently pull the lower eyelid away from the eye and install one drop in the conjunctival sac of the eye 3-5 times daily. Close your eyes slowly to allow an even distribution of the liquid on the eye surface. If the discomfort gets worse or does not improve, please consult your ophthalmologist.

### COMPOSITION

1 ml contains: sodium hyaluronate 0.15 %, Dextranthenol 2 %, sodium chloride, sodium citrate, citric acid, water for injections.

### PACKING

Commercial unit: Single dose containers, 20x0,5 ml

### PRECAUTIONS AND MARKINGS

\* Do not use the product if single dose container is damaged.

\* Do not touch the eye with the single dose container.

\* Do not use if you are allergic to any of the ingredients.

\* In very rare cases the application of the eye drops may cause irritation in very sensitive eyes.

\* For single use only: as HyloPan do not contain preservatives, any solution not used immediately after opening, should be discarded.

\* If you also use other eye drops / eye ointment, wait about 15 minutes. HyloPan should always be used last.

\* Do not use in infected or injured eyes.

\* Temporary blurring for a brief period may be experienced after application: avoid driving, operating machinery and wait until normal vision is restored.

\* Do not use the eye drops beyond the expiry date which you can find on each single dose container and on the folding box.

### STORAGE

Storage temperature: 2° to 25°C. Avoid freezing. Protect from light by storing the single dose containers in the folding box.

Lot-number and

Expiry date are embossed on each single dose container.

Aseptic manufacturing.

For description and mode of application see attached instructions for use

Storage temperature.

Do not use the product if single dose container is damaged.

For single use only.

Manufacturer:  
Penta Arzneimittel GmbH, Werksstrasse 3, D-92551 Stulln, Germany  
Manufacturer-Packer: Penta Arzneimittel GmbH, Werksstrasse 3, D-92551 Stulln, Germany

Zwittler Pharmaceuticals LTD  
Penitells Avenue 34-36  
Chalandri, Athens  
15234  
Greece

Date of information: 03/2017