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HYLOPAN® Lipo

DESCRIPTION

HYLOPAN® Lipo is especially suitable for the intensive symptomatic treatment of very dry and also chronically dry, irritated eyes. It cares for and moisturises the ocular surface and alleviates the symptoms of dry eyes (sicca syndrome, such as irritated, red, itchy, watery and/or very sensitive eyes or ocular surface) by forming a viscoelastic and clear protective film on the sensitive cornea without impairing visual performance. This is achieved by the high water-binding capacity of the sodium hyaluronate contained, the sodium salt of hyaluronic acid, which also occurs naturally in the vitreous body of the eye, among other things. In addition, **HYLOPAN® Lipo** contains provitamin B5 (Dexpanthenol), which has the ability to bind a high quantity of water, moisturizes and cares the eye, on the one hand, and on the other hand, supports the sodium hyaluronate to hold moisture. Furthermore, **HYLOPAN® Lipo** also contains vitamin A, vitamin E, phospholipids, carbomer and glycerine. The antioxidants vitamin A and vitamin E prevent the formation of free radicals and thus act as a protective complex for the cells on the surface of the eye.

Phospholipids, obtained from natural soya oil, support the regeneration of the natural lipid layer of the tear fluid. Carbomer and glycerine lead to a higher viscosity of the eye drops and thus ensure a longer retention time of the drops on the eye surface. This special eye drops composition with the above-mentioned ingredients protects and moisturises the surface of the eye, thereby noticeably and long-lastingly alleviating the symptoms of dry and chronically dry eyes.

HYLOPAN® Lipo effectively help with discomforts associated with:

- mechanical stress, e.g. during wearing of hard or soft contact lenses, or during ophthalmic diagnostic procedures
- environmental stress, e.g. induced by air conditioning, wind, coldness, dryness or air polluted by cigarette smoke for example
- strained eyes, e.g. work on the computer, microscopes and during/after long car journeys
- sand grain feeling
- use of certain medicines
- decreasing tear production with age
- allergens such as pollen, animal hair, dust, etc.

Preservative-free **HYLOPAN® Lipo** is also well-tolerated during long-term use. **HYLOPAN® Lipo** is suitable for contact lenses wearers. The contact lenses do not have to be removed during the application of the eye drops.

APPLICATION

Detach one single dose container. Make sure that each single dose container is intact (1). To open the single dose container, twist off the flat end (do not

pull) (2). Tilt your head back a little, look upwards and gently pull the lower eyelid away from the eye and instill 1-2 drops in the conjunctival sac of the eye 3-5 times daily (3). Close your eyes slowly to allow an even distribution of the liquid on the eye surface.

If the discomfort gets worse or does not improve, please consult your ophthalmologist!

FORMULATION

Milky white, sterile eye drop-emulsion.

COMPOSITION

Sodium hyaluronate, dexpanthenol, phospholipid, soy oil, vitamin E, vitamin A, oleic acid, glycerol, carbomer, citric acid, water for injections.

PACKAGE SIZES

Commercial units: Single dose containers 1x0.4ml/2x0.4ml/5x0.4ml/20x0.4ml/30x0.4ml/50x0.4ml
Not all pack sizes may be marketed.

PRECAUTIONS AND WARNINGS

- Do not use the product if single dose container/foil packaging (pouch) is damaged.
- Do not touch the eye with the single dose container.
- Do not use if you are allergic to any of the ingredients.
- In very rare cases, the use of the eye drops may cause irritation in very sensitive eyes or eye lids. The following disorders may occur: redness, pain, burning, stinging, itching, swelling, foreign body sensation.
- Do not use in infected or injured eyes.
- For single use only; as **HYLOPAN® Lipo** does not contain preservatives, any solution not used immediately after opening, should be discarded.
- If you also use other eye drops / eye ointment, wait about 15 minutes. **HYLOPAN® Lipo** should always be used last.
- Temporary blurring for a brief period may be experienced after application; avoid driving, operating machinery and wait until normal vision is restored.
- Do not use the eye drops for longer than 3 months after opening the pouch for the first time.
- Do not use the eye drops beyond the expiry date which you can find on each single dose container, on each foil packaging (pouch) and on each folding box.
- **HYLOPAN® Lipo** is filled into small plastic containers which could be swallowed by small children. Therefore, please keep the product out of the reach of children.
- In case a serious incident has occurred during the use of this medical device or as a result of its use, please report it to the manufacturer and the competent authority.

STORAGE

Store at 2° C to 25° C. Do not freeze. Protect from light by storing the single dose containers in the foil packaging (pouches) in the folding box.

EXPLANATION OF THE ICONS

Batch code

Expiry date

Date of manufacture

Sterilized using aseptic processing techniques

Single sterile barrier system with protective packaging outside

Consult instructions for use

Temperature limit and storage temperature

Keep away from sunlight, store the (single dose container/pouch) protected from light in the original folding box

For one single use only

Do not use the eye drops for longer than 3 months after opening the foil packaging (pouch) for the first time

Do not use if package is damaged (single dose container/pouch) and consult instructions for use

Medical device

Unique device identifier

Manufacturer

Distributor

0483

CE marking = symbol of free marketability in the EU with indication of the involved Notified Body

Manufacturer:

Penta Arzneimittel GmbH
Werksstraße 3
92551 Stulln, Германия

Distributor for Greece and Cyprus:

Zwitter Pharmaceuticals Ltd.
34-36 Pentelis Ave.
152 34 Chalandri, Athens
GREECE
T: +30 210 6821098
E: zwitter@zwitter.gr
W: www.zwitter.gr

CE 0483

Date of information: 03/2024